



The Mucat™ and Spirette™

Spirette™

This device is 23.5 cm long, 3.14 mm O.D. and consists of two parts:

- A flexible polypropylene sheath, with a beveled distal tip and depth marks at 4,7,8, and 10 cm
- An inner EVA piston that can slide via a soft acetal resin rod

Indications: Sampling

Ectocervical or endocervical sampling of the cervical mucus. After insertion through the cervical os, the Spirette™ tip is advanced to the sampling area: ectocervix or endocervix. Suction is generated by withdrawing the piston. Once the sample is collected, the Spirette™ is removed and the sample is smeared on a slide by pushing back the piston.

Mucat™

This device is 23.5 cm long, 3.14 mm O.D. and consists of two parts:

- A flexible polypropylene sheath, with a beveled distal tip
- An inner green EVA piston that can slide via a green soft acetal resin rod

Indications: Sampling

Ectocervical or endocervical sampling of the cervical mucus. After insertion through the cervical os, the Mucat™ tip is advanced to the sampling area: ectocervix or endocervix. Suction is generated by withdrawing the piston. Once the sample is collected, the Mucat™ is removed and the sample is smeared on a slide by pushing back the piston.

CCD Codes

1106000 Spirette™ (box of 25 units)
110600S Mucat™ Sterile (box of 25 units)

www.sepalreprodevices.com

410 Boston Post Road, Suite 21
Sudbury, Massachusetts 01776 USA
Email: info@sepalreprodevices.com
Main: 978.579.9389
Fax: 978.579.9407
Toll Free: 877.546.4223



Manufactured by Laboratoire C.C.D., Paris, France